
BiTrac Select™
Interchangeable
Standard Bronchoscopy
Elbow, 22mm Female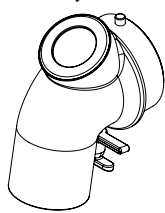

Operating Specifications:
5 °C to 40 °C at humidity range
of 15% to 95%

Storage Specifications:
-20 °C to 60 °C at relative
humidity up to 95% non-
condensing

Pulmodyne Inc.
2055 Executive Drive
Indianapolis, IN 46241 USA
www.Pulmodyne.com

EC REP
QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands

Intersurgical UAB
Tel + 370 387 66 611
e-mail: info@intersurgical.lt
Amionys g. 60, LT-18170, Palanga, Lithuania

Single Patient Use

Latex Free EN-Medical Device

Made in USA

Caution: Federal Law (USA) restricts this device
to sale by or on the order of a licensed physician.

Intended Use

The Interchangeable Standard Bronchoscopy Elbow is an accessory to the BiTrac Select Full Face Mask and the BiTrac Select MaxShield™ Mask, which allows CPAP and Bilevel therapy to be delivered to the patient during a bronchoscopy procedure. Use of this elbow with these masks is an accessory for use with CPAP and Bilevel systems used in the treatment of adult Obstructive Sleep Apnea (OSA) and/or ventilator support.

Note

The Interchangeable Standard Bronchoscopy Elbow is a temporary use elbow for use only during a bronchoscopy procedure.

Contraindications

1. This device should not be used on uncooperative or unresponsive patients.
2. This device may not be suitable for use on patients unable to remove the mask or are taking medication which may cause vomiting.

Warnings

1. This elbow should be used only during a bronchoscopy procedure while in the presence of trained medical personnel.
2. Use only with a ventilator equipped with adequate alarms and safety systems for ventilator failure. This elbow does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
3. This elbow does not have an integral leak port; therefore a separate exhalation device must be part of the circuit. Utilize the equipment manufacturer's recommended exhalation port in the designed circuit.
4. This device is designed to be used only in conjunction with a BiTrac Select Full Face or BiTrac MaxShield Select Interchangeable Elbow compatible mask.

Use of Interchangeable Elbow Feature:

1. Remove the existing elbow from the mask by holding the adapter ring and twisting the elbow counter clockwise to disengage the bayonet lock (Refer to Figure B.)
2. Gently pull the elbow from the mask. (Refer to Figure C.)
3. Gently press the desired Interchangeable Compatible Elbow into the mask adapter ring and turn clockwise until fully seated. Do not use excessive force.

Bayonet Lock References:

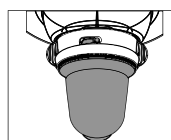
Figure A.

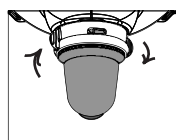
Figure B.


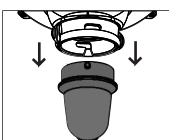
Figure C.

Pulmodyne is a registered trademark of Pulmodyne, Inc.
BiTrac Select and BiTrac Select MaxShield are trademarks of
Pulmodyne, Inc.

BiTrac Select™
Interchangeable
Standard Bronchoscopy
Elbow, 22mm Female

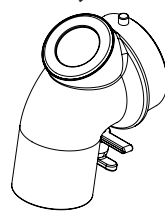Operating Specifications:
5 °C to 40 °C at humidity range
of 15% to 95%

Storage Specifications:
-20 °C to 60 °C at relative
humidity up to 95% non-
condensing

Pulmodyne Inc.
2055 Executive Drive
Indianapolis, IN 46241 USA
www.Pulmodyne.com

EC REP
QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands

Intersurgical UAB
Tel + 370 387 66 611
e-mail: info@intersurgical.lt
Amionys g. 60, LT-18170, Palanga, Lithuania

Single Patient Use

Latex Free EN-Medical Device

Made in USA

Caution: Federal Law (USA) restricts this device
to sale by or on the order of a licensed physician.

Intended Use

The Interchangeable Standard Bronchoscopy Elbow is an accessory to the BiTrac Select Full Face Mask and the BiTrac Select MaxShield™ Mask, which allows CPAP and Bilevel therapy to be delivered to the patient during a bronchoscopy procedure. Use of this elbow with these masks is an accessory for use with CPAP and Bilevel systems used in the treatment of adult Obstructive Sleep Apnea (OSA) and/or ventilator support.

Note

The Interchangeable Standard Bronchoscopy Elbow is a temporary use elbow for use only during a bronchoscopy procedure.

Contraindications

1. This device should not be used on uncooperative or unresponsive patients.
2. This device may not be suitable for use on patients unable to remove the mask or are taking medication which may cause vomiting.

Warnings

1. This elbow should be used only during a bronchoscopy procedure while in the presence of trained medical personnel.
2. Use only with a ventilator equipped with adequate alarms and safety systems for ventilator failure. This elbow does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
3. This elbow does not have an integral leak port; therefore a separate exhalation device must be part of the circuit. Utilize the equipment manufacturer's recommended exhalation port in the designed circuit.
4. This device is designed to be used only in conjunction with a BiTrac Select Full Face or BiTrac MaxShield Select Interchangeable Elbow compatible mask.

Use of Interchangeable Elbow Feature:

1. Remove the existing elbow from the mask by holding the adapter ring and twisting the elbow counter clockwise to disengage the bayonet lock (Refer to Figure B.)
2. Gently pull the elbow from the mask. (Refer to Figure C.)
3. Gently press the desired Interchangeable Compatible Elbow into the mask adapter ring and turn clockwise until fully seated. Do not use excessive force.

Bayonet Lock References:

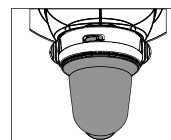
Figure A.

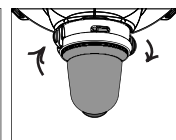
Figure B.

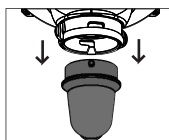
Figure C.

Pulmodyne is a registered trademark of Pulmodyne, Inc.
BiTrac Select and BiTrac Select MaxShield are trademarks of
Pulmodyne, Inc.

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