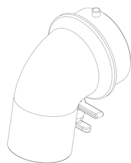
BiTrac MaxShield Select™

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Made in USA

Intended Use

The Interchangeable Standard Elbow is an accessory to the BiTrac MaxShield Select™ Pediatric Mask.

Contraindications

1. This device should not be used on uncooperative or unresponsive patients.
2. This device may not be suitable for use on patients unable to remove the mask or are taking medication which may cause vomiting.

Warnings

1. Use only with a ventilator equipped with adequate alarms and safety systems for ventilator failure. This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
2. This mask does not have an integral leak port; therefore a separate exhalation device must be part of the circuit. Utilize the equipment manufacturer's recommended exhalation port in the designed circuit.
3. This device is designed to be used only in conjunction with a BiTrac MaxShield Select Pediatric Interchangeable Elbow compatible mask.

Use of Interchangeable Elbow Feature:

1. Remove the existing elbow from the mask by holding the adapter ring and twisting the elbow counter clockwise to disengage the bayonet lock (Refer to Figure B.)
2. Gently pull the elbow from the mask. (Refer to Figure C.)
3. Gently press the desired Interchangeable Compatible Elbow into the mask adapter ring and turn clockwise until fully seated. Do not use excessive force.

Bayonet Lock References:

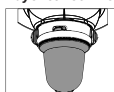
Figure A.

Figure B.

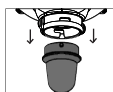
Figure C.

*European Union

Intersurgical UAB
Tel: +370 387 66 611
e-mail: info@intersurgical.lt
Amonio g. 60, LT-18170, Pabradė, Lithuania

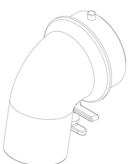
2745C 04-27-2022

Pulmonary is a registered Trademark of Pulmonary Inc. BiTrac MaxShield Select is a Trademark of Pulmonary Inc.

1 of 15

BiTrac MaxShield Select™

Pediatric Interchangeable Standard Elbow 22mm Female

Operating Specifications: 5°C to 40°C at humidity range of 15% to 95%

Storage Specifications: -20°C to 60°C at relative humidity up to 95% non-condensing

Medical Device
Single Patient Use

Caution: Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

Pulmonary Inc.
2055 Executive Drive
Indianapolis, IN 46241 USA
www.pulmonary.com

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