
BiTrac Vented™ Nasal Mask with Vented Elbow 22mm

Male

Latex Free

Single Patient Use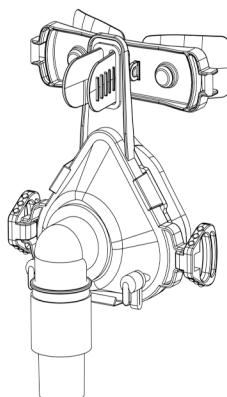

Caution:

Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

EC REP QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands


Made in USA

Operating Specifications: 5°C to 40°C at humidity range of 15% to 95%

Storage Specifications: -20°C to 60°C at relative humidity up to 95% non-condensing

Intended use: The nasal mask and accessories (mask, forehead pads and headgear) are used with and attached to a positive pressure air source, CPAP or BiLevel equipment, and has the Indications for Use consistent with Indications for Use of the equipment to which it is attached.

Typical Exhaust Flow / Pressure Performance of the Vent Swivel Elbow

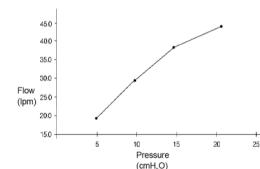Warnings:

1. This mask and vented swivel elbow has an integral leak port; therefore a separate exhalation device is not needed.
2. At low CPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
3. If skin irritation develops, this may be due to excessive pressure or skin reaction or a combination of the two. Loosen the head strap and/or try another mask or forehead pad size. Skin reaction may also be due to cleanliness. Refer to the cleaning instructions.
4. This CPAP mask should only be used with CPAP systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP system is turned ON and operating properly. The vent hole or holes associated with the mask should never be blocked.

Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP systems.

Nasal Mask Discomforts and Corrective Actions:

The problems listed below may be indications that your mask needs to be replaced. You should seek the assistance of your dealer when you notice one or more of the following:

- The cushion begins to tear
- The mask feels uncomfortable
- The parts show signs of wear, discoloration or cracking

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