BiTrac HC™

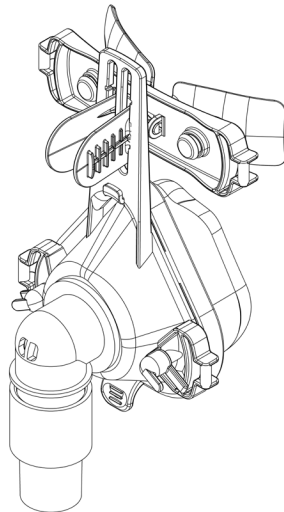
Nasal Mask with Vented Elbow 22mm Male

Latex Free

Single Patient Use Cleaning Instructions

Made in USA

Caution:

Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

Pulmodyne Inc.
2055 Executive Drive
Indianapolis, IN 46241 USA

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Cleaning Instructions:

Note: You do not have to remove the mask forehead pad or connector from the mask before cleaning it, but you may want to do so to clean the parts more thoroughly.

1. Remove the head strap from the mask. Disconnect the mask, connector and tubing.
2. With a soft cloth, gently wash the mask, forehead pad, head strap and connector in a solution of warm water and mild shampoo or dish detergent. DO NOT use any cleaners containing conditioners or moisturizers as these will leave a residue. Be sure to thoroughly clean the mask cushion flaps.
3. Rinse thoroughly and air dry only.
4. If the mask cushion or forehead pad still seem oily, wash them again.

Cleaning Warnings:

1. Do not Pasteurize
2. Do not gas, cold or steam sterilize.

Head Strap with Clip Cleaning Instructions:

1. Hand or machine wash in warm water with a mild detergent.
2. Do not use bleach, alcohol, cleaning solutions, moisturizers or conditioners.
3. Rinse thoroughly and air dry only.
4. Inspect for deterioration or damage and replace as needed.
5. Slight re-adjustment of clips on Head Strap after washing may be needed.

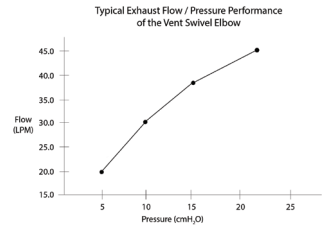
Operating Specifications: 5°C to 40°C at humidity range of 15% to 95%

Storage Specifications: -20°C to 60°C at relative humidity up to 95% non-condensing

Intended use: The nasal mask and accessories (mask, forehead pads and headgear) are used with and attached to a positive pressure air source, CPAP or Bilevel equipment, and has the Indications for Use consistent with Indications for Use of the equipment to which it is attached.

Warnings:

1. This mask and vented swivel elbow has an integral leak port; therefore a separate exhalation device is not needed.
2. At low CPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
3. If skin irritation develops, this may be due to excessive pressure or skin reaction or a combination of the two. Loosen the head strap and/or try another mask or forehead pad size. Skin reaction may also be due to cleanliness. Refer to the cleaning instructions.
4. This CPAP mask should only be used with CPAP systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP system is turned ON and operating properly. The vent hole or holes associated with the mask should never be blocked.

Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP systems.

Nasal Mask Discomforts and Corrective Actions:

The problems listed below may be indications that your mask needs to be replaced. You should seek the assistance of your dealer when you notice one or more of the following:

- The cushion begins to tear
- The mask feels uncomfortable
- The parts show signs of wear, discoloration or cracking

LIMITED WARRANTY

- Pulmodyne, Inc. warrants that the product will remain free from defects for a period of 3 months by proof of date of purchase from the original owner with lot number and return of the defective product.
- We do not offer a warranty if instructions are not followed, used or handled incorrectly, improper cleaning, negligence, normal wear and tear.
- This warranty does not cover cost for loaner and rental product, any transportation, any taxes or duty, and labor, and other related costs or other losses.
- Replacement of the defective part shall be the sole remedy.

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