

BiTrac Shield™ Size Guide



- 1.** Line the seal of mask below the bottom lip and flip the head gear over the patient's head.



- 2.** Fit the head gear around the top of the patient's head and bring the bottom clips to the front.



- 3.** Adjust the Head strap on the crown of the patient's head, while holding the BiTrac Shield in place.



- 4.** Attach both bottom clips from the head straps to the mask.



- 5.** Adjust the straps to fit:
Do Not Overtighten Head Straps



- 6.** Connect circuit and turn on flow. Re-adjust the head gear as needed.

