

BiTrac MaxShield™ Size Guide



1. Use the size guide before picking a mask. The mask should fit above the eyebrow and below the bottom lip.



2. Unwrap the mask from the packaging. Hold the head strap in front of the mask and rest the bottom of the mask below the bottom lip.



3. Flip the head strap over the patient's head and place the mask on the patient's face. Make sure the cushion is resting below the lower lip and above the eyebrow.



4. Adjust the head strap on the crown of patient's head while holding the mask in place.



5. Attach both bottom clips of the head strap to the mask. This should make an audible click sound.



6. Adjust top straps to fit:
Do Not Overtighten Head Straps



7. Once the top are done, adjust the bottom velcro straps.



8. Follow these steps again when the mask has been removed or refitting the mask.

***Model wears a Large.*

