



## BiTrac MaxShield Select™ Pediatric Size Guide

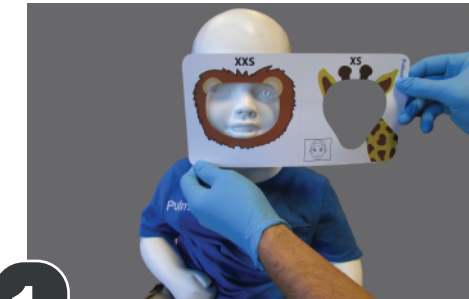

- 1.** Use the size guide provided in the case, to measure the patient's face.

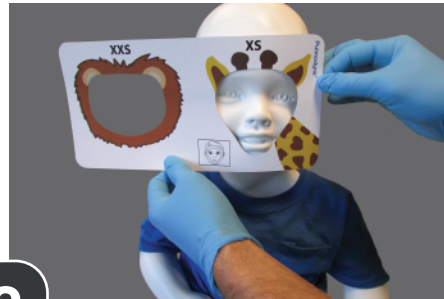

- 2.** The cut out should fit right above the patient's eyebrows, and right below the lower lip.



- 3.** Hold the correct size mask over the patient's face. Then slip the cap over the back of the patient's head.



- 4.** Secure the mask on the patient, by snapping the clips in place. There should be an audible click.



- 5.** Adjust the top velcro straps accordingly. Do not over tighten the mask.



- 6.** Once the top are done, then adjust the bottom velcro straps.



- 7.** After adjusting the straps, make sure the mask hasn't moved around. Check to make sure the cushion is above the eyebrow and below the lower lip.

*\*Mannquin is 1 year old to scale.*

