

Intended use:

While using these accessories please follow the Instructions For Use for the Aero-T. The Accessories listed in the chart below, are to be used only with the Aero-T. These devices can be used to aid in patient weaning and comfort.

Weaning with the O2-Vent

1. The O2-Vent can be used with the Aero-T by placing the Aero-T directly on the hub of the tracheostomy tube or inner cannula (dependent upon which Tracheostomy Tube you are using).
2. The O2-Vent should be connected onto the Aero-T if oxygen supply is needed. The oxygen will be humidified through the Aero-T and its concentration will be dependent upon flow.

Aero-T Holder

1. The Aero-T Holder should be placed directly onto the hub of the tracheostomy tube or inner cannula (dependent upon which Tracheostomy Tube you are using).
2. The Aero-T Holder helps aid in the replacement of an Aero-T, as it provides more comfort for patients that might have a sensitive tracheostoma.

Part Number Clean	Description
400102	Aero-T™ Holder
403102	O2-Vent
403202	O2-Vent Tube

