## BiTrac™ Circuit

### Bilevel Application

**Caution:**  
Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

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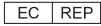
**Pulmodyne Inc.**  
2055 Executive Drive  
Indianapolis, IN 46241 USA

[www.pulmodyne.com](http://www.pulmodyne.com)

**EU\***

**Intersurgical UAB**  
Tel: + 370 387 66 611  
e-mail: [info@intersurgical.lt](mailto:info@intersurgical.lt)  
Armoniu g. 60, LT-18170, Pabradė, Lithuania

\*European Union

Made in the USA

## 72" Smooth ID Tube with attached filter, swivel connector with exhalation port

#### Intended Use

The BiTrac Circuit is used with and attached to a positive pressure air source, CPAP or Bilevel equipment, and has the Indications for Use consistent with the Indications for Use of the equipment to which it is attached. The circuit exhalation port also provides a continuous leak path in the patient circuit.

#### Warnings

1. The exhalation port is designed to vent CO<sub>2</sub> from the patient circuit. Continuous flow is required for safe operation. Do not occlude the exhalation port.
2. Before instituting mechanical ventilation, confirm the proper connection of the line circuit and verify ventilator operation. Patient, ventilator and circuit must be monitored on a regular basis according to local medical protocol.
3. Do not pull or stretch the circuit tubing. This could result in circuit leakage.
4. This circuit is not recommended for use with a heated humidifier since it does not allow for use of water traps.
5. This circuit is designed for **Single Patient Use**. It is not intended to be cleaned or sterilized. The length of time a circuit is used should be determined by local medical protocol.

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